Solitude Water Pillow

The principle of hydrostatic buoyancy provides natural support to the head and neck which aids in the elimination of muscle tension and pressure while maintaining correct sleeping posture.

- < The inner water cushion instantly adjusts and conforms to any head position providing maximum support and comfort while you sleep.
- < The removable water cushion is easily adjusted to your preference of soft, medium or firm support by simply adding the desired amount of water.

The Solitude Water Pillow is guaranteed* to be free from manufacturers' defects for 1 full year!

Filling instructions:

1. Remove water cushion from pillow.
2. Remove filling cap and add water.
3. Remove all air from the water cushion.
4. Replace filling cap securely.
5. Return water cushion to pillow pouch.
6. Burp the water cushion to reduce air bubbles and swishing.

The bladder can be warmed in a microwave, or cooled with a short stay in the freezer. *(Do not boil or freeze. This can cause serious harm to not only the pillow, but the user as well.)*

We recommend adding a water conditioner to prevent algae buildup during prolonged use or a teaspoon of bleach.

Firmness	Amount
Soft	2 litres
Medium	3 litres
Firm	4-5 litres

Precautions:

Do not use this pillow with electrical sleeping products such as electric blankets.

Do not use any sharp objects near the vinyl water cushion.

Do not drop the water cushion from heights greater than 1 metre.

Do not over fill the water cushion.

**Abnormal wear voids this warranty.*

Questions: Innotech Rehabilitation Products Inc.
PO Box 534, Orillia, Ontario L3V 6K2
1-800-361-0228
Visit us at: www.embraceairbackrests.com
Ask about our other products, the *EmbraceAirplus* air adjustable back support and *In-Line* seating.

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